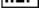
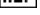
#3000 STRIP-ID

Pipette for oocytes manipulation

HOW SUPPLIED:

 #3000 Strip-ID125/10  #3000 Strip-ID130/10
 #3000 Strip-ID135/10  #3000 Strip-ID140/10
 #3000 Strip-ID145/10  #3000 Strip-ID150/10
 #3000 Strip-ID170/10  #3000 Strip-ID175/10
 #3000 Strip-ID200/10  #3000 Strip-ID275/10
 #3000 Strip-ID300/10  #3000 Strip-ID600/10

Important product information, read carefully before use.

Disposable – discard after single use

Re-use may have a negative influence on the development of the oocyte.

Caution: federal (USA) law restricts this device to sale by or on the order of a physician

Sterile: unless pouch is damaged or opened

DESCRIPTION

The Strip-ID is a single-use IVF laboratory micropipette intended for mechanical removal of the cumulus cell layers of the oocyte.

INDICATIONS FOR USE:

The Strip-ID is a micropipette used for oocytes manipulation.

CONTRAINDICATIONS:

There are no known contraindications for the use of this device.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

PRECAUTIONS:

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population.

The pipette should be used by healthcare professionals only.

1639

**Manufactured by:**

Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

PACKAGED:

- 10 pipettes / individual packaging.
- 15 individual packages in an inner box
- 5 inner boxes/shipping box

DIRECTIONS FOR USE:

1. Attach the pipette to a holder.
2. Aspirate the oocyte in and out the pipette until the cumulus cell layers are removed, the bigger diameters can be used for transfer or handling procedures of the oocyte.
3. Gently withdraw the pipette tip from the dish.
4. Remove the pipette from the holder and discard.
5. Discard in accordance with local regulation.

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